

Day 0

- 1. Each family member should have their own bag with an ORCHARDS Household Study Form inside.
 - The sheet is split into two portions: "TODAY" (Day 0) and "FOLLOW-UP" (Day 7)
 - Please fill out the top portion of the sheet ("TODAY", Day
 0) for each family member
 - TIP: Answering "no" to flu symptom question means the top portion of that form can remain blank
- 2. Collect a nasal swab specimen for each household member (excluding the child who participated in ORCHARDS).
 - Insert swab labeled with a GREEN dot into one nostril and rotate swab <u>3 times</u> against the nasal wall
 - Place the swab into the test tube of viral transport media labeled with a GREEN dot, snap off the excess swab shaft, & screw the cap on tightly
 - Place test tube back in small biohazard bag, and small biohazard bag back in the corresponding family member's bag
- 3. Place all family members' bags back into the large family bag, and place in the refrigerator (preferably in the middle of the fridge, to prevent freezing).

(Turn over for Day 7 directions!)

▶ Day 7

- 1. Today will be a repeat of Day 0.
 - Please fill out the bottom portion of each family member's ORCHARDS Household Study Form ("FOLLOW-UP", Day 7).
 - REMINDER: Child who participated in ORCHARDS will participate today.
- 2. Collect a 2nd nasal swab specimen for each household member (including the child who participated in ORCHARDS), following the same instructions as Day 0.
 - Use swab labeled with a RED dot
 - Place the swab into the test tube of viral transport media labeled with a RED dot, snap off the excess swab shaft, & screw the cap on tightly
 - Place test tube back in small biohazard bag, and small biohazard bag back in the corresponding family member's bag
- Place all family members' bags back into the large family bag, and place in the refrigerator until ORCHARDS staff comes to collect.
 - Please make sure you have signed all consent forms!

If you have questions at any point, please call the ORCHARDS study phone number: